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COVID-19 Ag

STANDARD™ Q COVID-19 Ag Test

PLEASE READ CAREFULLY BEFORE YOU PERFORM THE TEST

IVD Manufacture and Sales Approval Number 30300EZ00067000

REF 88018,88022

malcom

KIT CONTENTS

Test device (individually in a foil pouch with desiccant)

Extraction buffer tube

Nozzle cap


Sterilized swab

PREPARATION

1 Carefully read instructions for using the STANDARD Q COVID-19 Ag Test.

2 Check the expiry date at the back of the foil pouch. Do not use the kit, if expiry date has passed.

3 Check the test device and the desiccant pack in the foil pouch.

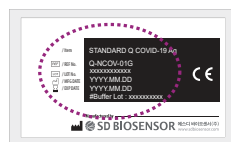<Foil pouch>

<Test device>

Yellow

Green

Yellow: Valid
Green: Invalid

<Desiccant>



How to use

Collect the nasal swab using a sterile swab

Collect nasopharyngeal swab using a sterile swab

1 Insert a sterile swab into the nostril of the patient, reaching the surface of the posterior nasopharynx.


2 Swab over the surface of the posterior nasopharynx.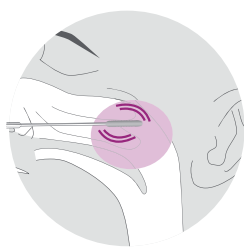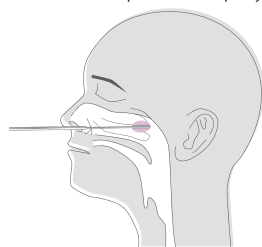

Collect the nasal swab using a sterile swab

1 Insert the swab about 2 cm along the nasal cavity.

2 Rotate the swab along the inner wall of the nose about 5 times. Leave it for about 5 seconds and pull it out.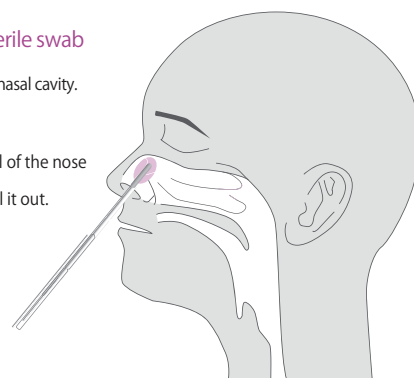


3 Insert the swab into an extraction buffer tube. While squeezing the buffer tube, stir the swab more than 5 times.

4 Remove the swab while squeezing the sides of the tube to extract the liquid from the swab.


5 Press the nozzle cap tightly onto the tube.

6 Apply 3 drops of extracted specimen to the specimen well of the test device.



7 Read the test result in 15-30 minutes.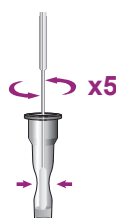

→

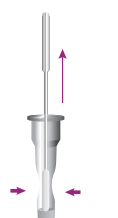→

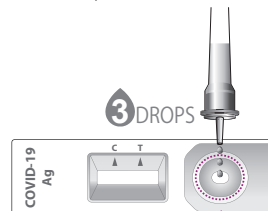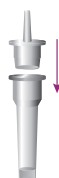
Read in
15-30 mins.
Do not read
after 30 mins.

Do not read test results
after 30 minutes.
It may give false results.

INTERPRETATION OF TEST RESULT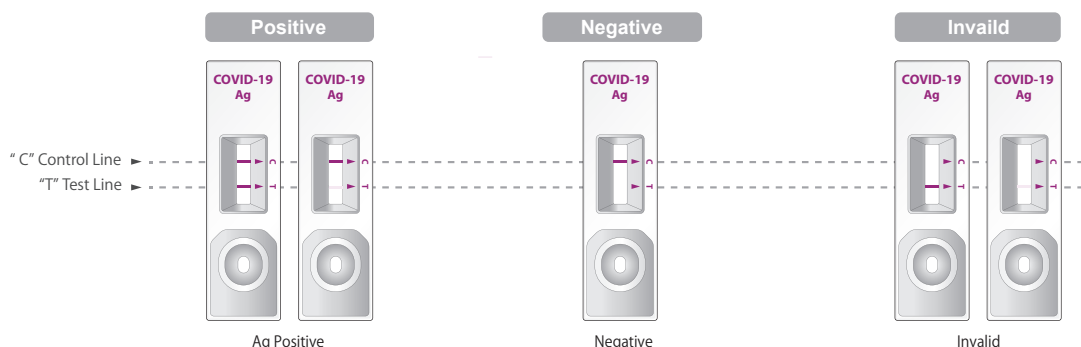

• If you get a positive result, you need to see a medical institution.
• Even if the result becomes negative, there is a risk of "false negative" that gives a negative result even though the virus is infected with a small amount, so it is necessary to continue infection control measures such as wearing a mask.

Manufacturer

SD BIOSENSOR

C-4&5Floor, 16, Deogyong-daero 1556beon-gil,
Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690,
REPUBLIC OF KOREA
Tel +82-31-300-0400 | Fax +82-31-300-0499

Manufacture and Sales

MALCOM CO.,LTD

15-10, HONMACHI 4-CHOME, SHIBUYA-KU, TOKYO,
151-0071, JAPAN
e-mail: mdinfo@malcom.co.jp
URL: <http://www.malcom.co.jp>